Sinonimia

MMN Medium

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- BHAT, P. & R.M. MYERS (1962) Standard methods and procedures used in bacteriology laboratory of the Vallore Christian Medical College Hospital for isolation and identification of organisms belonging to the family *Enterobacteriaceae*. Indian J. Med. Res. 50:559-566
- BHAT, P., S. SHANTHAKUMARI & H. ISAAC (1971) Mannitol-motility Medium in Routine Diagnostic Enteric Bacteriology. Indian J. Med. Res. 3:377-381
- D'AMATO, R.F. & K.M. TOMFOHRDE (1981) Influence of media on temperature-dependent motility test for *Yersinia enterocolitica*. J. Clin. Microbiol 3-11714(3):347-348
- GARD, S. (1938) Das Schwärmphänomen in der *Salmonella*-Gruppe und seine praktische Ausnützung. Z. Hyg. Infektionskr. 20:615-619
- LE MINOR, L. (1967) Le Diagnostic de Laboratoire des Enterobacteries. 3rd ed. Editions de la Tourelle. 94 Saint-Mande. Paris.
- MacFADDIN, J.F. (2000) Biochemical Tests for the Identification of Medical Bacteria. 3rd edition. Williams & Wilkins. New York.
- PICKETT, M.J. (1980) Non fermentative Gram-negative Bacilli: A syllabus for detection and identification. Scientific Developments Press. Los Angeles.
- ROLAND, P. & D. BOURBON (1949) Technique d'identification rapide des Enterobacteriacées. Ann. Inst. Pasteur 76:346-350
- TITSLER, R.P. & L.A. SANDHOLZER (1939). The use of semi-solid agar for the detection of bacterial motility. J. Bact. 31:575-580

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